









# BD Microtainer™ Tubes with Microgard™ Closure Tube Guide and Order of Draw

Catalog #/Closure Color      Additive      Mix by Inverting      Laboratory Use

 365974      Lavender	<b>FIRST</b>	K <sub>2</sub> EDTA	10x	For whole blood hematology determinations. Tube inversions prevent clotting.
 365965      Green		Lithium Heparin	10x	For plasma determinations in chemistry. Tube inversions prevent clotting.
 365985      Mint Green		Lithium Heparin and Gel for plasma separation	10x	For plasma determinations in chemistry. Tube inversions prevent clotting.
 365987      Mint Green				
 365992      Grey		NaFI/Na <sub>2</sub> EDTA	10x	For glucose determinations. Tube inversions ensure proper mixing of additive and blood.
 365967      Gold		Clot Activator and Gel for serum separation	5x	For serum determinations in chemistry.
 365978      Gold				
 365963      Red	<b>LAST</b>	No additive	0x	For serum determinations in chemistry, serology and blood banking.



365976  
Tube Extender

