

POINT OF CARE TESTING SUPPLIES

ITEM	MAUFACTURER	SUPPLY OBTAINED	COMMENTS
Urine Dipstick (Visual read or Clinitek Status)	Siemens	Hospital storeroom	ER/OB/IRMC- URGI. Multistix SG
Urine Dipstick QC (VISUAL READ)	Sentry- Visual read	Lab –Chem Frig # 2	Chestnut Ridge- UrgiCare
Automated Urine Control for Clinitek Status	Kenolor	Lab-Chem Frig #2	ER ,OB IRMC-URGICARE
Clinitek Status Printer Paper	Siemens	Lab- Gail office.Top bin of stack of yellow bins near file cabinet	ER,OB IRMC –URGICARE
Nitrazine Paper	Bristol Meyers	Lab – In box on top shelf at the Chem front counter	- If you issue the last box please leave note for gail.
Nitrazine QC – OB only	Fisher	Lab-Hematology -Low QC Fisher Buffer PH 4. - High QC- Fisher Buffer PH 7 - Labels attached to this list. No labels see next column - Notify Gail when you issue QC	If no labels info: - Have OB bring you the bottles they are currently using. Refill bottles. - Fill out label (blank specimen label) like example below & put on bottle Example : Nitrazine Low Qc Lot: _____ Exp: _____ In use: _____
Phydriion Paper-ER only	Microessential	*Special order by ER unit Manager. We have no supply in the lab.	

ITEM	MAUFACTURER	SUPPLY OBTAINED	COMMENTS
Phydriion QC	Fisher	Lab-Hematology -Low QC Fisher buffer PH4 - High Qc Fisher buffer PH 10. -- Labels attached to this list. No labels see next column - Notify Gail when you issue QC	If no labels info: - Have ER bring you the bottles they are currently using. Refill bottles. - Put new label (blank specimen label) on bottle. -Example of bottle label: See above Nitrazine.
Accu-Chek InformII	Roche Diagnostics	In Gail's Office.Issue meter In the right base first then move meter in left base to right base. - If have one meter issue a loaner. If no loaners try to borrow from another unit. - See next column for more info	-If no meters in Gail's office there are spares in the Classroom I&II. Located in the back of the classroom. Contact Nursing Education or Nursing Shift coordinator Or Security if Classroom I &II Is locked
Accu-chek Inform II Test Strip	Roche Diagnostics	Hospital Storeroom	
Accu-chek Inform II Controls (2levels) Level 1 = Low Level 2 = High	Roche Diagnostics	Hospital Storeroom	Controls pkg'd with Lo&Hi per Box. Have to be ordered as set.
Accu-chek InformII battery	Roche Diagnostics	N/A Batteries are Rechargeable via their base units	-Instruct them to place meter in base unit to charge
Surgilance Fingerstick devices -2 depths	Surgilance	Hospital Storeroom	
Heel stick Device-OB/NRSY only	Owens-Mumford	*Ordered by OB unit manager	

* Fecal Occult Blood Testing no longer done by nursing staff as of 1/29/08.
However as of 1/30/08, per Dr Wilson's request , boxes of testing supplies were placed on each unit as a courtesy for use by physicians only. How those supplies are replenished is up to each nursing unit. Supplies can be ordered from the hospital storeroom.

Nursing staff are not permitted to perform FOB tests as of 1/29/08. If a doctor writes orders for stools for occult blood nursing should send a stool sample to Lab .However if a doctor is on the unit and wants to perform a FOB from a rectal exam there is the courtesy box of supplies available.