




BEEBE HEALTHCARE ORDER OF DRAW TUBE CHART

<u>Tube 1</u>	<u>Tube 2</u>	<u>Tube 3</u>	<u>Tube 4</u>	<u>Tube 5</u>	<u>Tube 6</u>
<p>Blood Cultures Green/Orange =Set Yellow =Pediatric</p> 	<p>AFB/Fungal Blood Cultures Isolator Tube</p>  <p>**Tube stored in Microbiology</p>	<p>Waste Tube Used to clear air in line when using a butterfly.</p> 	<p>P2Y12/ASPRIN Platelet Function Top – White ring w/ black center</p> 	<p>Light Blue 2.7mL Clear Blue 1.8mL Sodium Citrate</p>  <p>** Requires Complete Fill ** Waste tube needed if only item to draw</p>	<p>Royal Blue 7.0mL Red Label Metal Free No Additive</p> 
<u>Tube 7</u>	<u>Tube 8</u>	<u>Tube 9</u>	<u>Tube 10</u>	<u>Tube 11</u>	<u>Tube 12</u>
<p>Plain Red 10.0mL No Additive</p> 	<p>Gold 3.5mL Serum Separator Tube with Gel</p> 	<p>Marble/Tiger 7.5mL Serum Separator Tube with Gel</p> 	<p>Light Green 3.5mL Lithium Heparin</p> 	<p>Dark Green 10.0mL Sodium Heparin</p> 	<p>Royal Blue 7.0mL Purple Label Metal Free Na2 EDTA</p> 
<u>Tube 13</u>	<u>Tube 14</u>	<u>Tube 15</u>	<u>Tube 16</u>	<u>Tube 17</u>	<u>Tube 18</u>
<p>Lavender 3.0mL K2 EDTA</p> 	<p>Pink 6.0mL K2 EDTA</p> 	<p>Black and Tan 10.0mL Cell-Free DNA BCT</p>  <p>** Waste tube needed if drawn directly behind a green top.</p>	<p>Gray 6.0mL Sodium Fluoride and Potassium Oxalate</p> 	<p>Yellow 6.0mL ACD Solution B</p> 	<p>Quantiferon Gold</p> <ol style="list-style-type: none"> 1) Gray 2) Green 3) Yellow 4) Purple  <p>** Waste tube needed if only item to draw</p>