

## Laboratory Specimen Collection Devices & Media

	<p><b>24 Hour Urine Container</b></p> <p>Orange: #457655 White: #450045</p>	<p>Check Test Catalog for preservative to add.</p> <p>Most specimens are refrigerated during collection and transport.</p> <p>White 24-hour container is for Uric Acid collection which is to remain AMBIENT (room temperature) during collection and transport.</p>
	<p><b>Urine Vacutainers</b></p> <p>Gray C&amp;S #457056 Yellow #450013</p>	<p><b>Gray Vacutainer for C&amp;S</b></p> <ul style="list-style-type: none"> <li>Urine Culture &amp; Sensitivity ONLY!</li> </ul> <p><b>Yellow tube (x2)</b></p> <ul style="list-style-type: none"> <li>Urinalysis ONLY (2 tubes needed)!</li> </ul> <p><b>Send 2 yellow tubes and 1 gray tube for "UA with culture"</b></p>
	<p><b>Stool Kit</b></p> <p>#713201119</p> <p>3 containers <i>Always check container label, do not go by color!</i></p>	<p><b>Cary Blair (pink liquid)</b> Enteric Pathogen, PCR</p> <p><b>SAF Fixative (clear liquid)</b> Ova &amp; Parasite (O&amp;P)</p> <p><b>Empty Vial (plain container)</b> Adenovirus C-diff Toxin (liquid stool only) Lactoferrin Occult Blood Immunoassay (FIT) Rotavirus</p>
	<p><b>Sterile Container</b></p> <p>#850516</p>	<p>Body Fluid Sputum Tissue (fresh) <b>Urine Clean Catch Midstream</b> Urinalysis Urine C&amp;S</p> <p><b>Urine First Void</b> APTIMA Urine (CT, GC, Trich) Chlamydia Only PCR Urine Gonococcus Only PCR Urine Chlamydia &amp; Gonococcus PCR Urine</p>
	<p><b>eSwab</b></p> <p>Standard Tip (White) #450094</p> <p>Mini Tip (Blue) #450093</p>	<p>Anaerobic Culture Bordetella Pertussis PCR Ciprofloxacin Screen Fungal culture (Oral/Genital) Gram Stain (BV and Yeast) (only for ages 10 - 60 years old) Group A Strep Throat Culture</p> <p>Group B Strep MRSA Screen Respiratory (non-sputum) Culture Throat Culture Comprehensive Trichomonas Antigen (&gt;10 years) Wound Culture</p>
	<p><b>UTM</b></p> <p>With regular flocked swab ("Viral Transport Media")</p> <p>#652889</p>	<p>Flu A/B PCR Herpes Culture Herpes Lesion PCR Mycoplasma/Ureaplasma Culture Respiratory Virus Film Array RSV Rapid</p> <p>Viral Culture (Indicate Source)</p>
	<p><b>Abbott Multi-collect Kit</b></p> <p>#659762</p>	<p>Chlamydia + Gonococcus PCR Genital Swab or Urine Chlamydia Only PCR Genital Swab or Urine Gonococcus Only PCR Genital Swab or Urine</p> <p>For testing Chlamydia (CT), Gonococcus (GC), Trichomonas (Trich.) on &gt;14 years old. For patients &lt;14 years old, please use Aptima collection for send out testing.</p>
	<p><b>APTIMA</b></p> <p>Endocervical Swab Kit #301041 (white)</p> <p>Multitest Kit (Vaginal) #301162 (orange)</p> <p>Urine #301040 (yellow)</p>	<p>Aptima Chlamydia + Gonococcus Aptima Chlamydia Only Aptima Gonococcus Only Aptima Trichomonas</p> <p>These tests cannot be added to a previous Pap test order.</p> <p>For testing Chlamydia (CT), Gonococcus (GC), Trichomonas (Trich.) on &gt;14 years old. Patients &lt;14 years will be sent to outside lab for testing (Reference Miscellaneous testing)</p>
	<p><b>Thin Prep Container</b> <i>Not available via Lawson. Order via Supply Order Form to Lab</i></p> <p>Pick brush <b>OR</b> broom</p>	<p>Aptima Chlamydia + Gonococcus Aptima Chlamydia Aptima Gonococcus Aptima Trichomonas</p> <p>Pap Test (Indicate HPV Reflex) HPV High Risk Screen HPV 16/18/45 genotype</p> <p>The only lubricating jellies to be used with the ThinPrep® PAP test are the Aseptic Control PAP Test Lubricating Jelly (#653380) and Surgilube® (#657243).</p>

**Always use 2 Patient Identifiers (name and date of birth) on the specimen and on the order. Never use nicknames.**

Always make sure that lids to containers are on correctly and tight! Leaky specimens will be rejected.

**Please direct questions to the Call Center at 616.774.7721. Laboratory Specimen Collection Catalog: [spectrumhealth.testcatalog.org](http://spectrumhealth.testcatalog.org)**

**GENERAL TEST LISTING, NOT ALL TESTS ARE LISTED.** Information correct as of May, 2018. For up-to-date information consult the lab specimen collection catalog.