
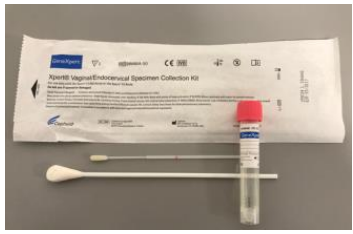
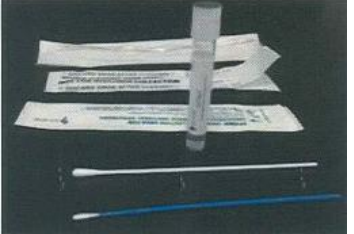




The Sexual Assault Kit contains the following containers / swabs, please contact Microbiology Lab for more kits (x72585).

Urine Cup	GeneXpert Vaginal/Endocervical Specimen Collection Kit	APTIMA Unisex Swab Specimen Collection Kit	APTIMA Multitest Swab Specimen Collection Kit	BD ESwab Collection and Transport System
				

Please collect the following swabs for STI testing according to the specimen source:

		Test		
		Chlam trachomatis and N.gonorrhoeae PCR <u>CT GC PCR</u>	Chlam trachomatis and GC Aptima UW PCR <u>GCCTAD</u>	Culture, GC Screen + Wet Mount <u>C GCS+WM</u>
Specimen Source	Urine, Dirty Catch 1 Urine Cup	Collect one cup of Dirty Catch Urine for all Three Tests		
	Genital (Vaginal) 3 Types of Swabs	GeneXpert Vaginal/Endocervical Specimen Collection Kit	APTIMA Multitest Swab Specimen Collection Kit	BD ESwab Collection and Transport System
	Genital (Penile) 3 Types of Swabs	GeneXpert Vaginal/Endocervical Specimen Collection Kit	APTIMA Multitest Swab Specimen Collection Kit	BD ESwab Collection and Transport System
	Genital (Urethral) 3 Types of Swabs	GeneXpert Vaginal/Endocervical Specimen Collection Kit	APTIMA Unisex Swab Specimen Collection Kit	BD ESwab Collection and Transport System
	Genital (Endocervical) 3 Types of Swabs	GeneXpert Vaginal/Endocervical Specimen Collection Kit	APTIMA Unisex Swab Specimen Collection Kit	BD ESwab Collection and Transport System
	Throat 3 Types of Swabs	GeneXpert Vaginal/Endocervical Specimen Collection Kit	APTIMA Multitest Swab Specimen Collection Kit	BD ESwab Collection and Transport System
	Rectal 3 Types of Swabs	GeneXpert Vaginal/Endocervical Specimen Collection Kit	APTIMA Multitest Swab Specimen Collection Kit	BD ESwab Collection and Transport System

If clinically indicated for Herpes (HSV1 and 2), call lab for COPAN Flocked Swab with UTM to collect for HSV Type 1 and 2 Qualitative PCR Test (HSV Rapid PCR, Swab) from vaginal or penile sites.