

URINE COLLECTION GUIDE




Lawson # 288100

BD Vacutainer Collection Kit

- Patients collect urine samples in urine cup.
- Clinic laboratory coworkers transfer urine to Vacutainer urine tubes. **Send BOTH tubes filling gray Urine C&S tube to fill line.**
- Send Vacutainer urine tubes to Mercy Laboratory.

Send urine cup **only** if short sample is obtained and cannot be transferred to Vacutainer urine tubes.

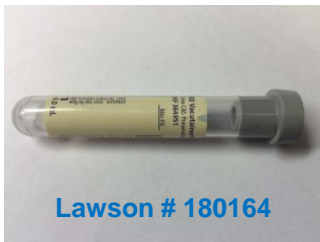

Lawson # 180164

- ✓ **Urine Culture**
MUST be filled to fill line!



Lawson # 108131

- ✓ **Urinalysis (UA)**
OR
- ✓ **Urine Chemistries***

NOTE: Submit 2 yellow tubes if UA and Urine Chemistries (i.e., Microalbumin, Creatinine, Protein, etc.) are requested.



- ✓ **Urinalysis with Reflex Culture**
OR
- ✓ **Urine Culture**
AND
- Urinalysis (UA)**



- ✗ **Please DO NOT send conical urine tubes for Urine Chemistries. Require round bottom tubes.**