

**Respiratory Pathogen Testing?  
So many SWABS!  
START HERE**

BBL culture swab  
(double red cap)  
PS# 0500356



← Group A Strep

**WHAT AM I TESTING FOR?**

Respiratory Pathogen Panel (RPP) →

BD Universal Viral Transport  
PS# 5017049



Influenza (FLU) and/or  
Respiratory Syncytial Virus (RSV)

**ARE YOU PERFORMING POINT OF CARE OR SENDING IT TO A LAB?**

POC

Sending to a lab

**FLU OR RSV**

**WHERE ARE YOU LOCATED?**

FLU

RSV


WCH, UH or clinic with lab services

MOI or clinic without lab service


Use the foam tipped nasal swab included in kit



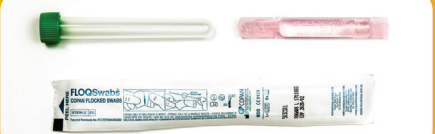
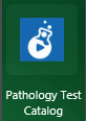
Use the Hydraflock Sterile flocked nasopharyngeal swab included in kit



You can use one swab for both RSV and FLU:  
Copan FLOQ nasopharyngeal swab  
PS# 5027950



Use a FLOQ nasopharyngeal swab. Place in 1 ml saline in sterile tube  
Swab PS# 5019619  
Sterile Tube PS# 5007415  
Saline PS# 5003758

Always refer to the Pathology Test Catalog at [muhealth.testcatalog.org](http://muhealth.testcatalog.org) found on Citrix receiver or in Power Chart for the most update-to-date testing information.

*For convenience, e-procure numbers have been added to this poster for ordering purposes.*